

Happier patient
detected.

Aligner undetected.



SureSmile® Clear Aligners

Improved treatment experiences detected.

SureSmile® Clear Aligners can help provide your patients with the smile they want. And now with SureSmile® VPro™, and SureSmile® Retainer, you can offer your patients more reasons to choose SureSmile® Clear Aligners.*



SureSmile® Clear Aligners— the clear aligner solution from Dentsply Sirona

Through the course of treatment and beyond, you have the support from Dentsply Sirona with over 100 years in oral healthcare. SureSmile® has more than 20 years of experience providing orthodontic treatment solutions. When you choose SureSmile® Clear Aligners, you can rest assured knowing you have the backing of decades of dental experience.


*Talk to your Dentsply Sirona Specialist for market availability.

▶ You determine the treatment goal.
We provide the plan.

With the user-friendly SureSmile® Treatment Planning software, it's simple to upload digital impressions, X-rays, and patient photos from almost any source. You can rely on our Digital Lab Specialists to do the rest and send you an optimised treatment plan. Or, if you prefer, you can customise your own preferences. Our open platform fully integrates with all Dentsply Sirona scanners, as well as other most commonly used scanners.


▶ Review anytime, anywhere.

When the Digital Lab has created the initial treatment plan, you'll receive it for your review and approval. The initial setup is organised and easy. The SureSmile® Aligner software is intuitive and offers a thorough review of each patient's case with a wide range of different tools. Our software is browser-based, so you can review your treatment plans whether you're at your practice, at home, or traveling.

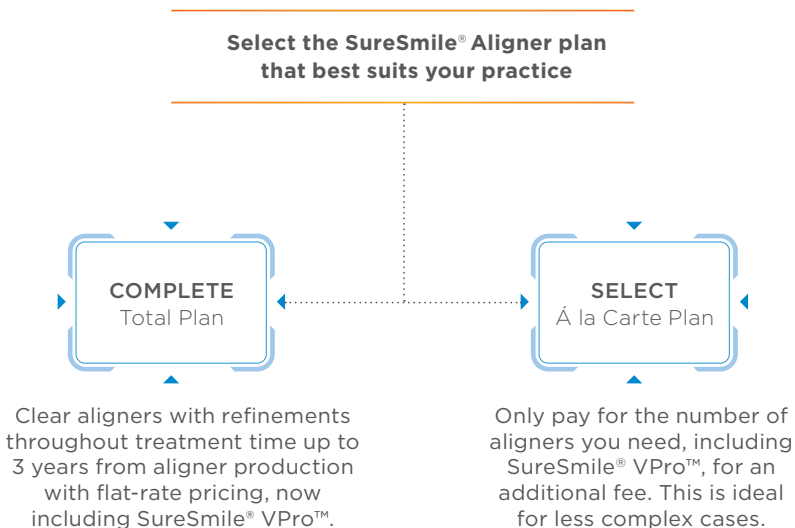

🔍 Clinical excellence detected.

SureSmile® Clear Aligners are made by experts using advanced technology and proven materials. Unique features allow for personalisation for even the most challenging cases. Precise processing of the data and thorough quality control ensure outstanding fit, excellent durability, and aligners so transparent that they're practically undetectable.

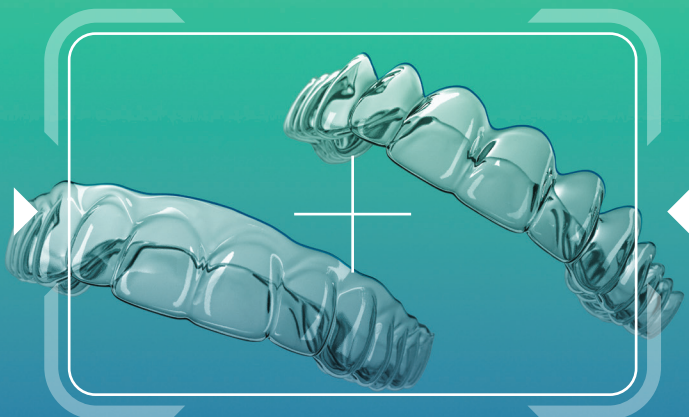

Comfort comes in a variable trimline


With SureSmile® Clear Aligners, you can customise the design of your trimline. An automated, high-precision, laser-cutting process is used to offer the ultimate in comfort. SureSmile® Aligner edges are exceedingly smooth, whether you choose a **straight or scalloped trimline**.

The straight trimline follows the gingiva at a consistent distance, which you determine. A small overlap of the gingiva can have a positive effect on the force transmission and even reduce the number of necessary attachments. The scalloped trimline follows the course of the gingiva and is even less noticeable than the straight trimline.




Experience 50% fewer refinements than with other clear aligners¹

What happens when innovative technology is combined with highly advanced materials? The kind of customised fit that provides the ultimate comfort for your patients—from first placement through their final treatment week. Each SureSmile® Clear Aligner undergoes rigorous quality checks and is validated by a clinical expert from our Digital Lab to ensure an accurate treatment plan for your patients. The result? A clear aligner system that fits into a patient's lifestyle with 50% fewer refinements than other clear aligners and can improve the doctor/patient relationship.

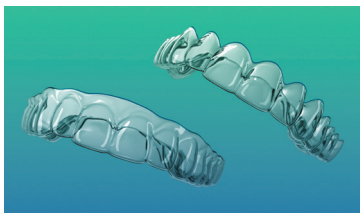

The SureSmile® Aligner provides excellent force transmission

SureSmile® Clear Aligners incorporate high performance materials to handle multiple attachments and are customisable.

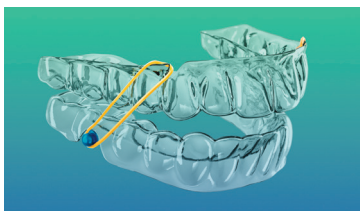


Correction of complicated cases

SureSmile® Clear Aligners are adept at helping you treat more complex cases. Slits in the aligner allow the insertion of intermaxillary elastics, and larger cut-outs provide access to buttons that are attached to the teeth.



Smiles by design

When X-ray data are provided, they are registered with a 3D model to superimpose photos in order to plan the final tooth position to be in line with the natural facial features of your patient.

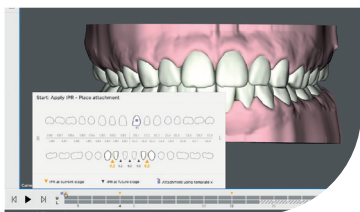

Timelines and IPR scheduling

Our software shows a graphic animation of your patient's treatment plan. The timeline indicates when attachments have to be placed and at which appointments interproximal enamel reduction (IPR) has to be performed and on which teeth.

Reference: 1. SureSmile® Aligner Refinement Data. Data on file 2020. Keim/Vogels/Vogels, 2020 JCO Study of Orthodontic Diagnosis and Treatment Procedures (JCO/October 2020, p. 581 et seq.)

SureSmile®

Dentsply
Sirona

Seamless integration detected.

We've taken our experience in digital planning for orthodontic treatment, combined with our experience as a dental solutions company, and provided you with a simple way to conduct initial setups and treatment plans that result in predictable patient outcomes and compelling results.

The SureSmile® Treatment Planning Workflow can be smoothly and seamlessly integrated into any practice. Our software is browser-based, which eliminates the need to download directly to your computer. We follow these steps:



Collect records

Photos and a digital impression are taken at the first appointment. SureSmile® Treatment Planning Workflow accepts scans from most commonly used intraoral scanners. Optional for clear aligner treatment: X-ray data for a more comprehensive diagnosis.




Submit the prescription

Rely on our Digital Lab Specialist or specify your detailed treatment preferences.



Approval or modification of treatment plan

All treatment plans submitted to the SureSmile® Digital Lab go through a comprehensive planning process and a final quality inspection by a Digital Lab Specialist. This ensures excellent initial setups.

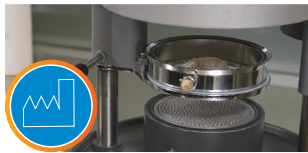

Manufacturing the aligners

Once you've approved the treatment plan, SureSmile® Clear Aligners are custom fabricated using an innovative process.



Starting treatment

When the Patient Kit, SureSmile® VPro™ and SureSmile® Clear Aligners arrive at your practice, it's time to start treatment.



Finishing treatment

SureSmile® Retainers are essential to ensuring happy patients' new smiles stay straight.

Happy patients detected.

- 1 Help patients achieve their best smile with SureSmile® VPro™—a 5-minute, at-home device that uses gentle vibrations to accelerate orthodontic tooth movement, which may reduce discomfort and may shorten treatment time.¹⁻³



Helping your patients protect their investment

Don't forget to order the SureSmile® Retainer designed to prevent teeth relapse and protect patients' investment in their smile.

The SureSmile® Aligner Training Program

We are in this together and will partner with you every step of the way. The SureSmile® Aligner Training Program is an eLearning resource available to you at your convenience on our award-winning platform, SureSmileU, where you'll also find guides, videos, job aids, and live monthly training events with expert clinicians. Upon completion of the training program, you'll be a SureSmile Aligner provider. The training program consists of 5 steps and can be completed in just 90 minutes:

STEP 1

CASE SELECTION



10 minutes

STEP 2

RECORDS



20 minutes

STEP 3

SOFTWARE NAVIGATION



35 minutes

STEP 4

SETUP & STAGING



15 minutes

STEP 5

NEXT STEPS QUIZ



10 minutes

References: 1. Alansari S, Atique MI, Gomez JP, et al. The effects of brief daily vibration on clear aligner orthodontic treatment, *J World Fed Ortho*. 2018;7(4):134-140. doi:10.1016/j.ejwf.2018.10.002 2. Shipley T. Effects of high frequency acceleration device on aligner treatment—a pilot study. *Dent J (Base)*. 2018;6(3):32. doi:10.3390/dj6030032 3. Nicozisis J, Brigham G, Sparaga J, Shipley T. Effects of 120Hz high-frequency acceleration device on orthodontic discomfort. *Preprints*. Preprint posted online October 24, 2018. doi:10.20944/preprints201810.0557.v1

SureSmile®

SureSmile®

Cover image represents post treatment results.

ANZ  clientservices@dentsplysirona.com  11-21 Gilby Road, Mount Waverley, VIC 3149  www.dentsplysirona.com

Australia  1300 552 929 | **New Zealand**  0800 336 877

YOUR
CONFIDENCE
IS OUR
INSPIRATION.

 **Dentsply
Sirona**